

Accessories for ultrasonic baths



Insert basket



Inset basket



Cassette holder



Silicone knob mat

Intended purpose

The products support ultrasonic cleaning by holding and positioning the objects to be cleaned. The products are used in conjunction with BANDELIN ultrasonic baths in the context of manual reprocessing, and before or after mechanical reprocessing of medical devices.

The products are class I medical devices according to Regulation (EU) 2017/745.



User group

The products are used in commercial reprocessing by qualified specialists. Observe the instructions for use of the preparation and the ultrasonic bath

Safety



Risk of burns

The products may be hot during or after cleaning and reprocessing.

- Contact with the products may cause burns.
- Wear suitable protective gloves or ensure

that the products have cooled down sufficiently before removing them from the ultrasonic bath.



Risk of infection

As part of their intended use, the products come into contact with contaminated instruments. There is a risk of infection when handling them.

- Wear suitable protective gloves and, if necessary, other personal protective equipment.
- Do not use damaged products.



Risk of injury

Damaged products could have sharp edges and cause injuries.

- Check regularly for damage.
- Do not use damaged products.



Product damage

The use of aggressive agents or mechanical overload can reduce the lifespan of the products.

- Check regularly for damage.
- Do not use damaged products.
- Avoid overloading the products. Observe the maximum load capacity.


Product	Maximum load capacity
K 08, K 1 C	1 kg
K 3 C, K 3 CL	2 kg
K 5 C, K 6, KAH 14.3	5 kg
K 6 L, K 6 BL, K 10, K 10 B, K 14, K 14 B, K 28, K 28 C, K 40, K 50, K 50 C	10 kg



For more information, visit www.bandelin.com.

Reprocessing instructions



The products must be cleaned before each use and before disposal.


Serious incidents must be reported to BANDELIN electronic GmbH & Co. KG and to the competent authorities.





www.manuals.bandelin.com





info@bandelin.com
☎ : +49 30 768 80-0
📠 : +49 30 773 46 99